
Turon


MedTech

en: Instruction for use
bg: Инструкция за употреба
cs: Návod k použití
da: Brugsanvisning
de: Gebrauchsanweisung
el: Οδηγίες χρήσης
es: Instrucciones de uso
et: Kasutusjuhend
fi: Käyttöohjeet
fr: Instructions d'utilisation
hr: Upute za upotrebu
hu: Használati utasítás
it: Istruzioni per l'uso

lt: Naudojimo instrukcija
no: Bruksanvisning
lv: Lietošanas instrukcija
nl: Instructies voor gebruik
pl: Instrukcja użytkowania
pt: Instruções de utilização
ro: Instrucțiuni de utilizare
sk: Návod na použitie
sl: Navodila za uporabo
sv: Bruksanvisning
ru: Инструкция по применению
tr: Kullanım talimatı
uk: Інструкція із застосування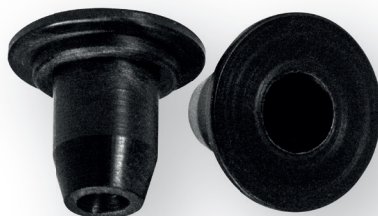

Turon MedTech AB
Kardanvägen 21 • 43232 SE-Varberg • Sweden
www.turonmedtech.com

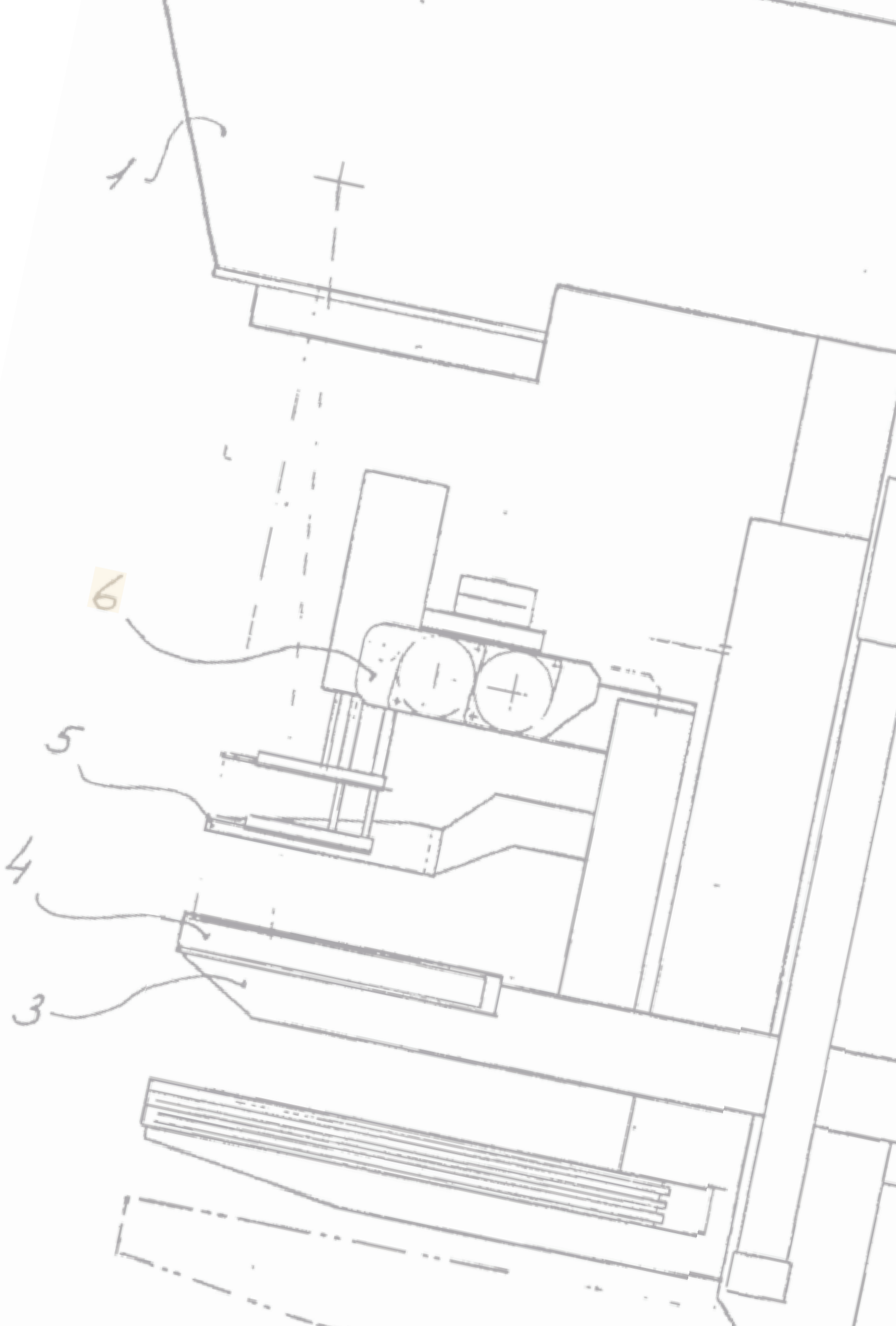English	04
Български език	05
Česky	06
Dansk	07
Deutsch	08
Ελληνικά	09
Español	10
Eesti	11
Suomi	12
Français	13
Hrvatski	14
Magyar	15
Italiano	16
Lietuvių	17
Latviešu	18
Nederlands	19
Norsk	20
Polski	21
Português	22
Română	23
Русский	24
Slovenčina	25
Slovenščina	26
Svenska	27
Türkçe	28
Українська	29

English - Instruction for use

Needle guide bushings

Intended use

These bushings are designed to guide a biopsy needle and create a sterile protection when performing biopsy in mammography equipment. Bushings are a medical device.

Instructions for use

- Verify that bushings are compatible with the biopsy equipment, e.g., consult biopsy equipment manufacturer or biopsy equipment instructions for use.
- Bushings shall be fixated in a needle guide.
- Bushings are delivered sterile. (Method: Ethylene oxide sterilization)
- Bushings are intended for single use only. If bushings are re-used they will no longer be sterile.
- If the sterile packaging is damaged bushings shall not be used and shall be disposed.
- The general hygiene guidelines and legal regulations for the proper disposal of potentially infectious substances of human origin should be considered and followed.
- If any serious incident occurs in relation to the device, the incident shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

REF

	Gauge
040301	Ga 18
040302	Ga 20
040307	Ga 14
040308	Ga 16
040309	Ga 20 open
040311	Ga 22
040312	Ga 12

Explanation of symbols

CE Mark

Catalogue number

Sterilized using ethylene oxide

Manufacturer

Do not re-use

Batch code

Date of manufacturer

Use-by date

Example of bushings fixated in needle guide.

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